

# BASIS High

Setting a New Industry  
Standard

ACRYLIC



- Ultra High Impact
- Superb Color Stability & Resistance to Flexure Fatigue
- 40-Minute Heat Cure



BASIS HIGH



[www.ydp-usa.com](http://www.ydp-usa.com)  
[www.yamahachidentalusa.com](http://www.yamahachidentalusa.com)  
Phone: (877) 320-2909  
[info@yamahachidentalusa.com](mailto:info@yamahachidentalusa.com)

yamahachi   
Dental Products USA

BASIS HIGH is a Brand distributed by Yamahachi Dental Products USA Inc.



## BASIS HIGH MAIN FEATURES

- Ultra High Impact
- Superb Color Stability & Resistance to Flexure Fatigue
- 40-Minute Heat Cure



## ACRYLIC

# BASIS HIGH TECHNICAL INSTRUCTIONS

1.	Apply small amount of separator (Apollon Sep) evenly to plaster model and allow to dry.
2.	Make a diatoric hole to improve retention.
3.	<p>Measuring and Mixing:</p> <ul style="list-style-type: none"> <li>a) Measure 10% more powder than the weight of the provisional wax base.</li> <li>b) Mix the powder and liquid well. Mixing ratio: Powder/Liquid=100g/43ml. Put it into a plastic bag and mix well to prevent any bubbles forming, Cover for 10 minutes at 23C (73F).</li> <li>c) When resin has reached doughy state fill the mould.</li> </ul>
4.	Fill the model with the doughy resin and press it. Use a polyethylene sheet to avoid touching the resin directly. Final pressure will be performed without the sheet. Leave the filler for 30 minutes at 23C (73F).
5.	<p>Curing and cooling:</p> <ul style="list-style-type: none"> <li>a) Place the flask into water and heat the water to boiling (approx. 30 minutes). Boil for 30-40 minutes and cure resin completely.</li> <li>b) Remove the flask and cool at room temperature for approx. 30 minutes. Then put the flask into water until completely cooled. Remove denture from flask.</li> </ul>
6.	Polish as usual.
7.	Place the cured denture base in water and allow remaining monomer to dissolve before placing in mouth.

Shade Guides Available Upon Request.

<https://yamahachidentalusa.com/products/basis-hi-acrylic-resin>

Package information:

Powder: 1 Kilo and 3 Kilos

Shades: O Pink, LF Pink, V Pink

  
**BASIS HIGH**



www.ydp-usa.com  
www.yamahachidentalusa.com  
Phone: (877) 320-2909  
info@yamahachidentalusa.com

**yamahachi**  
Dental Products USA 

BASIS HIGH is a Brand distributed by Yamahachi Dental Products USA Inc.